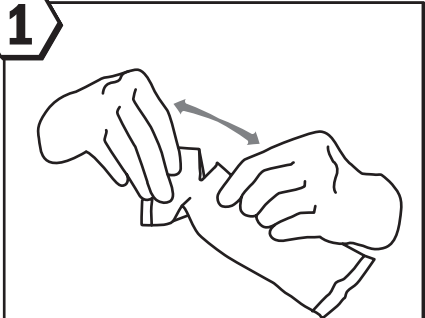


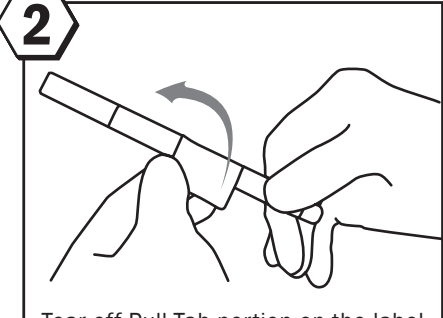
KWIK-STIK™ devices contain a lyophilized pellet of a single strain of microorganism or a defined mixed population of microorganisms. The selection of KWIK-STIK™ microorganisms supports quality assurance programs in microbiology laboratories providing clinical diagnostic services and a wide variety of testing services.

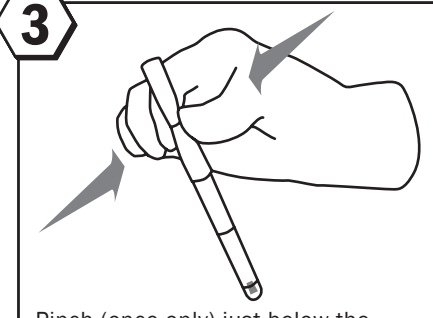


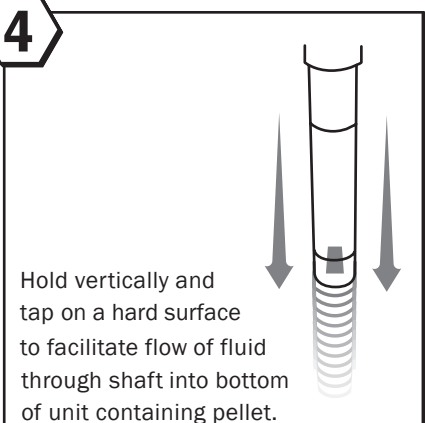
KWIK-STIK™

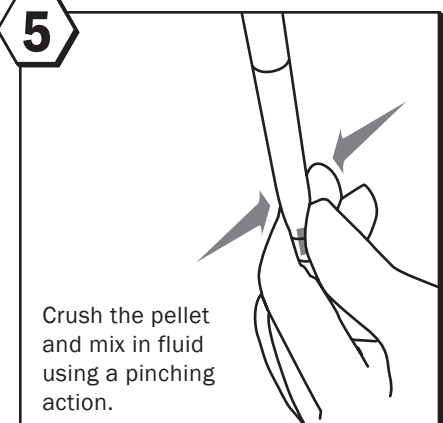
Also for KWIK-STIK™ Plus & EZ-COMPT™
Simply Efficient

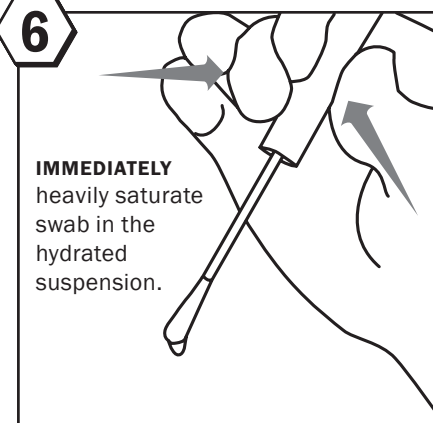
- 

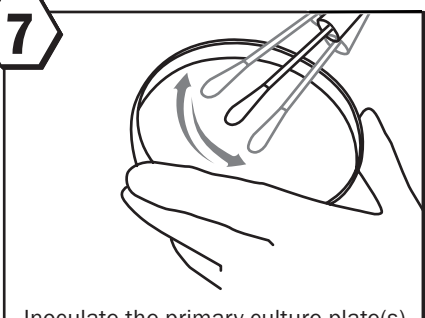
1 Tear open pouch at notch and remove the KWIK-STIK™.
- 

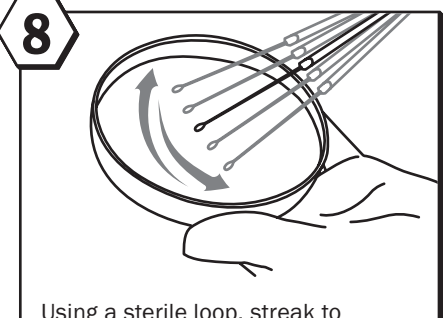
2 Tear off Pull-Tab portion on the label and attach it to the primary culture plate or QC record.
- 

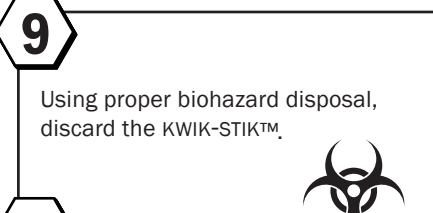
3 Pinch (once only) just below the fluid meniscus of the ampule found in the cap to release the hydrating fluid.
- 

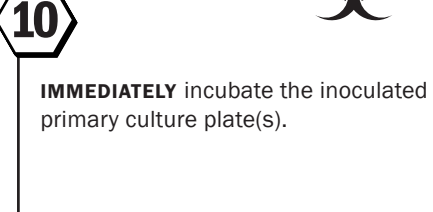
4 Hold vertically and tap on a hard surface to facilitate flow of fluid through shaft into bottom of unit containing pellet.
- 

5 Crush the pellet and mix in fluid using a pinching action.
- 

6 **IMMEDIATELY** heavily saturate swab in the hydrated suspension.
- 

7 Inoculate the primary culture plate(s) by using pressure and rolling the swab over one-third of the plate.
- 

8 Using a sterile loop, streak to facilitate colony isolation.
- 

9 Using proper biohazard disposal, discard the KWIK-STIK™.
- 

10 **IMMEDIATELY** incubate the inoculated primary culture plate(s).